

# Instru Zym Bio-wipes

## Product information



### Product description

Instru Zym Bio-wipes are ready-to-use enzyme-based cleaning wipes. The wipes are made of 100% biodegradable fabric with optimum active ingredient delivery.

### Area of application

They are used for pre-cleaning of medical devices. Coarse, organic soiling is removed gently.

### Instructions for use

Clean the instrument thoroughly with the wipe as soon as possible after use. If necessary, repeat with a new wipe until the surface is visually clean. Make sure surfaces are completely wetted. Pre-cleaning should preferably be carried out with the wipes from the clean end of the instrument to the soiled end. For professional use only.

### Product range and specification

Item number	Container size	Container type	Packaging unit
600897	80 wipes	flowpack	6 x 80 wipes



# Instru Zym Bio-wipes

## Product information



### Medical device class

I

### Composition

Protease, amylase, lipase, cellulase, non-ionic surfactants, preservatives, fragrances, purified water

### Chemicophysical data

**Colour** slightly yellowish  
**Appearance** clear  
**State** liquid  
**pH-value (20°C)** 5,5 - 6,5  
**Density (20°C)** 0,9982 - 1,0082 g/ml

### Storage and shelf life

**Shelf life**  
 24 months  
**Shelf life after opening**  
 28 days  
 Carefully reclose container after each use.

### Storage environment requirements

Store the product tightly closed in a well-ventilated location and protect it against unauthorised access. Observe the following storage temperature limits.

Description	Unit	Value
Temperature limits for short-term storage (min. - max.)	[°C]	0 - 40 <sup>1)</sup>
Ambient temperature for long-term storage according to shelf life (min. - max.)	[°C]	20 - 25

<sup>1)</sup> Short term storage (max. 24 h) up to max. 40 °C has no effect on the effectiveness of the product.

### Disposal

Disposal must be carried out in accordance with applicable national and local regulations. Please also note the information from the Material Safety Data Sheet.

### Information

Further information is available on the internet on our homepage:  
[www.drdeppe.de](http://www.drdeppe.de)

Special enquiries should be addressed to:  
[info@drdeppe.de](mailto:info@drdeppe.de)

