

Lotio 2in1

Product information

Product description

Lotio 2in1 are ready-to-use disinfectant wipes for hand and surface disinfection containing aloe vera. The CO₂ neutral wipe material is 100% biodegradable.

Area of application

One wipe for two applications: Lotio 2in1 can be used for both hand disinfection and surface disinfection.



Spectrum of activity

- bactericidal (incl. MRSA), yeasticidal
- active against enveloped viruses (incl. HBV, HIV, HCV, vaccinia-, influenza- and corona-viruses)

Instructions for use

Clean very soiled hands and surfaces before disinfecting. To disinfect hands, moisten the dry hands with the moist wipe and keep them moist for the entire exposure time, using a second wipe if necessary. For surface disinfection, wet the surface completely and wipe with a second cloth if necessary. The surface must be kept moist for the entire contact time. Observe the exposure times.

Product range and specification

item number	container size	wipe size	container type	packaging unit	EAN/GTIN (each)	EAN/GTIN (packaging unit)
601643	60 wipes	18 x 20 cm	flowpack	20 x 60 wipes	4260181254722	4260181254739



DrDeppe

Lotio 2in1

Product information

Biocide registration number

N-86407

Use disinfectant safely. Always read the label and product information before use.

Composition

The active ingredients contained in 100 g soaking solution are:

0,8 g Didecyldimethylammonium chloride

0,5 g 2-Phenoxyethanol

Storage and shelf life

Shelf life

24 months

Shelf life after opening

28 days

Carefully reseal the container after each use.

Only use wet wipes.

Store the product tightly closed in a well-ventilated location and protect it against unauthorised access.

Exposure times

Lotio 2in1	condition		norm	exposure time
	dirty	clean		
bactericidal		x	EN 13727* EN 16615	1 min.
yeastocidal		x	EN 13624* EN 16615	
active against enveloped viruses (Modified Vaccinia Ankara)		x	EN 14476*	

* tested on soaking solution

Informations

about this product



Further information is available on our homepage
www.drdeppe.de

Special enquiries should be addressed to
info@drdeppe.de



Laboratorium Dr. Deppe GmbH
Hooghe Weg 35 · 47906 Kempen
T +49 (0) 2152 5565-0 W www.drdeppe.de



DrDeppe