

Disposal of disinfectants

General indications:

In general, large residual quantities or unopened containers of disinfectants must be disposed of properly. Disposal should be carried out by an appropriate disposal company. For this purpose, the respective product safety data sheet must be made available to the disposal company concerned.

Residual amounts of disinfection concentrates:

Small quantities, such as residues from used containers, can be added to the regular waste water after thoroughly diluting with cold tap water in a ratio of 1:20. When handling undiluted disinfectants, always observe the warning and safety instructions on the product label or in the product information and take protective measures if necessary.

Residual quantities of working solutions from the daily demand:

Residues of prepared ready-to-use solutions can be added under running water to the regular waste water.

Residual amount of ready-to-use disinfectants:

Residues and small quantities of ready-to-use disinfectants can be added under running water to the regular waste water. When handling undiluted disinfectants, always observe the warning and safety instructions on the product label or in the product information and take protective measures if necessary.

Disinfection wipes:

Disinfectant wipes soaked with alcoholic disinfectants may still contain small residual amounts of the alcoholic active ingredients (ethanol, propanol) after use.

Single wipes can be disposed of in household waste without hesitation after use, as long they are not mixed with hot, glowing or burning waste.

Larger quantities of used wipes should be temporarily stored in a well-ventilated room in a large open plastic container or bucket until the wipes have dried completely. Care should be taken that the temporarily stored wipes will not come into contact with glowing objects (e.g. flames, sparks, cigarette ash). After drying out, the wipes can be disposed of with household waste.

Your Laboratorium Dr. Deppe Team



Laboratorium Dr. Deppe GmbH
Hooghe Weg 35 · 47906 Kempen
T +49 (0) 2152 5565-0 W www.drdeppe.de