

# Spray On wipes

## Product information

### Product description

Spray On wipes are alcohol-based surface disinfectant wipes in ready-to-use form, aldehyde and QAV-free. The CO<sub>2</sub> neutral wipe material is 100% biodegradable.

### Area of application

Health care facilities and pharmaceutical and cosmetic industry, for example patient-near surrounding, working areas/desks, general equipment (excluding food contact surfaces): disinfection of small hard/non-porous surfaces.

### Spectrum of activity

- bactericidal (incl. MRSA), yeasticidal
- active against enveloped viruses (incl. HBV, HIV, HCV, vaccinia-, influenza- and corona-viruses)

### Instructions for use

The surface to be disinfected is wiped and wetted with a sufficient amount of the product to ensure complete wetting. The surface to be disinfected must be kept moist for the entire contact time in order to achieve effectiveness.



The biocidal product Spray On wipes (active substance Propan-1-ol and Propan-2-ol) has a Union authorisation for product-type 2 (disinfectants and algacides not intended for direct application to humans or animals) and product-type 4 (food and feed area) with the authorisation number EU-0027668-0000. The Union authorisation is valid until 31 July 2032.

## Product range and specification

item number	container size	wipe size	container type	packaging unit	EAN/GTIN (each)	EAN/GTIN (packaging unit)
601212	80 wipes	20 x 20 cm	flowpack	12 x 80 wipes	4260181253879	4260181251318
601214	70 wipes	29 x 30 cm	bucket	1 x 70 wipes	4260181253893	4260181251332
601213	70 wipes	29 x 30 cm	bucket-refill pack	6 x 70 wipes	4260181253886	4260181251325

# Spray On wipes

Product information

## Union authorisation number

EU-0027668-0000

Use disinfectant safely. Always read the label and product information before use.

## Composition

**The active ingredients contained in 100 g soaking solution are:**

45 g Propan-1-ol

15 g Propan-2-ol

## Storage and shelf life

### Shelf life

flowpack 24 months,

bucket and bucket refill pack 36 months

### Shelf life after opening

28 days

Carefully reseal the container after each use.

Only use wet wipes.

Store the product tightly closed in a well-ventilated location and protect it against unauthorised access.

## Exposure times

Spray On wipes		condition		norm	exposure time
		dirty	clean		
bactericidal	with mechanical action	x		EN 13727*	1 min.
				EN 16615	
yeastcidal		x		EN 13624*	
				EN 16615*	
active against enveloped viruses (Modified Vaccinia Ankara)		x		EN 14476*	
				EN 13697	

\* tested on soaking solution

## Informations

about this product



Further information

will be available on

our homepage

[www.drdeppe.de](http://www.drdeppe.de)

Special enquiries

should be addressed to

[info@drdeppe.de](mailto:info@drdeppe.de)



**Laboratorium Dr. Deppe GmbH**

Hooghe Weg 35 · 47906 Kempen

T +49 (0) 2152 5565-0 W [www.drdeppe.de](http://www.drdeppe.de)