

Instru Star rfu

Product information

Product description

Instru Star rfu is a ready-to-use, aldehyde-free instrument disinfectant.

Area of application

It is suitable for the disinfection of medical, surgical instruments using the immersion method. It is also suitable for rotary dental instruments (drills). Not suitable for final disinfection of semicritical instruments.



Spectrum of activity

- bactericidal (including MRSA), yeasticidal
- tuberculocidal, mycobactericidal
- active against enveloped viruses (including HBV, HIV, HCV, vaccinia-, influenza-, and corona viruses)
- virucidal effective against Polyoma SV-40

Instructions for use

Immersion method

Immerse instruments immediately after cleaning in the undiluted liquid. All surfaces and cavities must be covered or filled. Please respect the contact times. Remove instruments, rinse thoroughly with water, and dry. Do not insert any aluminium parts. Follow the hygiene requirements for the reprocessing of medical devices (in Germany: RKI, KRINKO, MPBetreibV). Follow the instructions of the instrument and equipment manufacturers. For professional use only.

Product range and specification

item number	container size	container type	packaging unit	EAN/GTIN (each)	EAN/GTIN (packaging unit)
601490	2 L	canister	6 x 2 L	4260181252407	4260181252414
600091	5 L	canister	2 x 5 L	4260181253947	4260181251363
600071	10 L	canister	1 x 10 L	4260181253930	4260181251356



Instru Star rfu

Product information

Medical device class

IIb

Composition

The active ingredients contained in 100 g:
1.125 g cocosporylene diammonium borate

Storage and shelf life

Shelf life

36 months

Shelf life after opening

12 months

Close the container carefully after each use.

Store the product in a well-ventilated place in a sealed condition and protect it from unauthorized access.

Exposure times

Instru Star rfu	condition		norm	exposure time
	dirty	clean		
bactericidal	x		EN 13727	15 min.
			EN 14561 / DGHM	
yeasticidal			EN 13624	
			EN 14562 / DGHM	
tuberculocidal (M. terrae)			EN 14348	
			EN 14563 / DGHM	
mycobactericidal (M. terrae, M. avium)			EN 14348	
			EN 14563 / DGHM	
active against enveloped viruses (Vaccinia strain Elstree)			EN 14476	
virucidal effective against Polyoma SV-40	10% FBS		DVV/RKI	

Information

about this product



For further information, please visit our website
www.drdeppe.de

For specific inquiries, please contact
info@drdeppe.de

CE 0044



Laboratorium Dr. Deppe GmbH

Hooghe Weg 35 · 47906 Kempen

T +49 (0) 2152 5565-0 W www.drdeppe.de



DrDeppe