

Instru Suc

Product information

Product description

Instru Suc is a universal, economical, low-foaming, and aldehyde-free concentrate for disinfecting suction units.

Area of application

Instru Suc is used for disinfecting suction units with and without amalgam separators.

Spectrum of activity

- bactericidal (including MRSA), yeasticidal

Instructions for use

Biofilm prevention

Daily disinfection with a 2% working solution can effectively prevent biofilm formation.

Biofilm removal

Existing biofilms can also be removed with Instru Suc: After 16 hours of contact time (overnight) with a 2% working solution, no germs were detected for an existing biofilm.

For professional use only.



Product range and specification

item number	container size	container type	packaging unit	EAN/GTIN (each)	EAN/GTIN (packaging unit)
601489	2 L	canister	6 x 2 L	4260181252421	4260181252438
600047	5 L	canister	2 x 5 L	4260181250700	4260181251295

Instru Suc

Product information

Medical device class

IIb

Composition

The active ingredients contained in 100 g:
7.5 g Dioctyldimethylammoniumchloride

Storage and shelf life

Shelf life

36 months

Shelf life after opening

12 months

Close the container carefully after each use.

Store the product in a well-ventilated place in a sealed condition and protect it from unauthorized access.

Exposure times

Instru Suc	condition		norm	concentration	exposure time
	dirty	clean			
bactericidal	x		EN 13727 / VAH EN 14561 / VAH 15	2%	15 min.
yeastocidal	x		EN 13624 / VAH EN 14562 / VAH 15		

Information

about this product



For further information, please visit our website
www.drdeppe.de

For specific inquiries, please contact
info@drdeppe.de

CE 0044



Laboratorium Dr. Deppe GmbH

Hooghe Weg 35 · 47906 Kempen

T +49 (0) 2152 5565-0 W www.drdeppe.de