

# Spray In QF

Product information



## Product description

Spray In QF is a ready-to-use, liquid, alcoholic, aldehyde and QAV-free surface disinfectant.

## Area of application

For rapid disinfection of alcohol-resistant surfaces in patient environments, clinics, practices, outpatient departments, and care areas, as well as in industry and public facilities. Excluded are surfaces of instruments, semicritical and critical medical devices.

Spray In QF is also suitable for individuals in the food sector, certified as HACCP-compliant according to EU Regulation (EC) No. 852/2004, and can be safely used within the framework of an HACCP concept.

## Spectrum of activity

- bactericidal (incl. MRSA), yeasticidal
- tuberculocidal, mycobactericidal
- active against enveloped viruses (incl. HBV, HIV, HCV, vaccinia-, influenza- and corona-viruses)
- limited spectrum of virucidal activity (incl. noro-, rota-, adeno-viruses)

## Instructions for use

The ready-to-use product is preferably to be used in the wiping method using a disposable wipe. Spray disinfection should preferably be limited to areas that are not reachable by wiping disinfection. When using the spray method, it should be combined with wiping method. It is important to keep the surface to be disinfected moist throughout the entire contact time to achieve effectiveness.



## Product range and specification

item number	container size	container type	packaging unit	EAN/GTIN (each)	EAN/GTIN (packaging unit)
601356*	500 ml	bottle	18 x 500 ml	4260181254043	4260181251196
600917	1 L	club bottle	12 x 1 L	4260181254012	4260181251165
601106	5 L	canister	2 x 5 L	4260181254029	4260181251172
601107	10 L	canister	1 x 10 L	4260181254036	4260181251189

\* item is a biocidal product

# Spray In QF

Product information



## Medical device class

IIa (except for item no. 601356 - biocidal product only)

## Biocide registration number

N-73614

Use disinfectant safely. Always read the label and product information before use.

## Composition

The active ingredients contained in 100 g:  
50 g Ethanol

## Storage and shelf life

### Shelf life

36 months

### Shelf life after opening

12 months

Close the container carefully after each use.

Store the product in a well-ventilated place in a sealed condition and protect it from unauthorized access.

## Exposure times

Spray In QF		condition		norm	expo- sure time	
		dirty	clean			
bactericidal	with mecha- nical action		x	EN 13727/ EN 1276 EN 16615*	1 min.	
yeasticidal			x	EN 13624/ EN 1650 EN 16615*		
active against enveloped viruses (Modified Vaccinia Ankara)			x	EN 14476 virus-4-field- test**		
bactericidal	without mecha- nical action		x	EN 13727/ EN 1276 EN 13697		
yeasticidal			x	EN 13624/ EN 1650 EN 13697		
active against enveloped viruses (Modified Vaccinia Ankara)			x	EN 14476 EN 16777		
limited spectrum of virucidal activity (Adenovirus type 5, Murine Norovirus)			x	EN 14476		
tuberculocidal (M. terrae)			x	EN 14348		
mycobactericidal (M. terrae und M. avium)		x		EN 14348		5 min.

\* tested on reference wipe

\*\* according to EN 16615

## Information about this product



For further information,  
please visit our website  
[www.drdeppe.de](http://www.drdeppe.de)

For specific inquiries,  
please contact  
[info@drdeppe.de](mailto:info@drdeppe.de)

CE 0044



**Laboratorium Dr. Deppe GmbH**  
Hooghe Weg 35 · 47906 Kempen  
T +49 (0) 2152 5565-0 W [www.drdeppe.de](http://www.drdeppe.de)