

# Spray Off

## Product information

### Product description

Spray Off is a liquid ready-to-use, alcoholic and aldehyde-free surface disinfectant.

### Area of application

For the disinfection of alcohol-resistant surfaces in the patient environment, in clinics, practices, outpatient departments, and care areas, as well as in industry and public facilities. Excluded are surfaces of instruments, semi-critical and critical medical devices.

### Spectrum of activity

- bactericidal (incl. MRSA), yeasticidal
- fungicidal
- active against enveloped viruses (incl. HBV, HIV, HCV, vaccinia-, influenza- and corona-viruses)
- limited spectrum of virucidal activity (incl. noro-, rota-, adeno-viruses)
- virucidal effective against Polyoma SV-40
- virucidal

### Instructions for use

The ready-to-use product is preferably to be used in the wiping method using a disposable wipe. Spray disinfection should preferably be limited to areas that are not reachable by wiping disinfection. When using the spray method, it should be combined with wiping method. It is important to keep the surface to be disinfected moist throughout the entire contact time to achieve effectiveness.



### Product range and specification

item number	container size	container type	packaging unit	EAN/GTIN (each)	EAN/GTIN (packaging unit)
600986	500 ml	bottle	18 x 500 ml	4260181253794	4260181250762
601330	1 L	club bottle	12 x 1 L	4260181254357	4260181250779
601334	5 L	canister	2 x 5 L	4260181254371	4260181250786
601332	10 L	canister	1 x 10 L	4260181254388	4260181250793

# Spray Off

Product information

## Medical device class

IIa

## Biocide registration number

N-91345

Use disinfectant safely. Always read the label and product information before use.

## Composition

The active ingredients contained in 100 g:

63.50 g Ethanol

0.05 g Didecyldimethylammonium chloride

## Storage and shelf life

**Shelf life**

36 months

**Shelf life after opening**

12 months

Carefully reclose container after each use.

Store the product tightly closed in a well-ventilated location and protect it against unauthorised access.


## Exposure times

Spray Off		condition		norm	expo- sure time	
		dirty	clean			
bactericidal	with mechani- cal action	x		EN 13727 EN 16615*	1 min.	
yeastocidal		x		EN 13624 EN 16615*		
tuberculocidal / mycobactericidal (M. terrae, M. avium)		x		EN 14348		
fungicidal (C. albicans, A. brasiliensis)		x		EN 13624 EN 13697	5 min.	
active against enveloped viruses according to DVV/RKI (Vaccinia strain Elstree, BVDV strain NADL)			10% FBS	DVV/RKI	1 min.	
active against enveloped viruses according to EN 14476 (Modified Vaccinia Ankara)				x	EN 14476	1 min.
limited spectrum of virucidal activity according to DVV/RKI (Murine Norovirus, Adenovirus type 5)			10% FBS	DVV/RKI	1 min.	
virucidal effective against Polyoma SV-40			10% FBS	DVV/RKI	5 min.	
virucidal (Murine Norovirus, Adenovirus type 5, Poliovirus type 1)				x	EN 14476 EN 16777	5 min.

\* tested on reference wipe

### Information

about this product

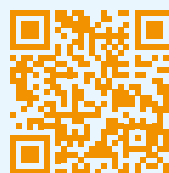

For further information,  
please visit our website  
[www.drdeppe.de](http://www.drdeppe.de)

For specific inquiries,  
please contact  
[info@drdeppe.de](mailto:info@drdeppe.de)

CE 0044



**Laboratorium Dr. Deppe GmbH**

Hooghe Weg 35 · 47906 Kempen

T +49 (0) 2152 5565-0 W [www.drdeppe.de](http://www.drdeppe.de)