

Spray On wipes

Product information

Product description

Spray On wipes are ready-to-use, alcoholic surface disinfectant wipes, free from aldehydes and QACs. The CO₂-neutral wipe material is made from renewable raw materials and is 100% biodegradable.

Area of application

Health care facilities and pharmaceutical and cosmetic industry, for example patient-near environments, working areas/desks, general equipment (excluding food contact surfaces): disinfection of small hard/non-porous surfaces.

Spectrum of activity

- bactericidal (incl. MRSA), yeasticidal
- active against enveloped viruses (incl. HBV, HIV, HCV, vaccinia-, influenza- and corona-viruses)

Instructions for use

The surface to be disinfected is wiped and sufficiently wetted with the product to ensure complete coverage. It is essential to keep the surface to be disinfected moist throughout the entire exposure times to achieve effectiveness.



The biocidal product Spray On wipes (active ingredients: Propan-1-ol and Propan-2-ol) has received Union and Swiss authorizations for product type 2 disinfectants and algacides not intended for direct application to humans or animals) and product-type 4 (food and feed area) with authorization numbers EU-0027668-0000 and CH-0031680-0000. The Union authorization is valid until July 31, 2032.

Product range and specification

item number	container size	wipe size	container type	packaging unit	EAN/GTIN (each)	EAN/GTIN (packaging unit)
601212	80 wipes	20 x 20 cm	flowpack	12 x 80 wipes	4260181253879	4260181251318
601214	70 wipes	29 x 30 cm	bucket	1 x 70 wipes	4260181253893	4260181251332
601213	70 wipes	29 x 30 cm	bucket-refill pack	6 x 70 wipes	4260181253886	4260181251325

Spray On wipes

Product information

Union authorisation number

EU-0027668-0000

Swiss authorisation number

EU-0027668-0000

Use disinfectant safely. Always read the label and product information before use.

Composition

The active ingredients contained in 100 g soaking solution:

45 g Propan-1-ol

15 g Propan-2-ol

Storage and shelf life

Shelf life

flowpack 24 months,

bucket and bucket refill pack 36 months

Shelf life after opening

28 days

Close the container carefully after each use.

Only use moist wipes.

Store the product in a well-ventilated place in a sealed condition and protect it from unauthorized access.

Exposure times

Spray On wipes		condition		norm	exposure time
		dirty	clean		
bactericidal	with mechanical action	x		EN 13727*	1 min.
				EN 16615	
yeastcidal		x		EN 13624*	
				EN 16615	
active against enveloped viruses (Modified Vaccinia Ankara)		x		EN 14476*	

* tested on soaking solution

Informations

about this product



For further information, please visit our website

www.drdeppe.de

For specific inquiries, please contact

info@drdeppe.de



Laboratorium Dr. Deppe GmbH

Hooghe Weg 35 · 47906 Kempen

T +49 (0) 2152 5565-0 W www.drdeppe.de