

# Instru Star rfu

## Product information

### Product description

Instru Star rfu is a ready-to-use, aldehyde-free instrument disinfectant.

### Area of application

It is suitable for the disinfection of medical, surgical instruments using the immersion method. It is also suitable for rotary dental instruments (drills). Not suitable for final disinfection of semicritical instruments.



### Spectrum of activity

- bactericidal (including MRSA), yeasticidal
- tuberculocidal, mycobactericidal
- active against enveloped viruses (including HBV, HIV, HCV, vaccinia-, influenza-, and corona viruses)
- virucidal effective against Polyoma SV-40

### Instructions for use

#### Immersion method

Immerse instruments immediately after cleaning in the undiluted liquid. All surfaces and cavities must be covered or filled. Please respect the contact times.

Remove instruments, rinse thoroughly with water, and dry. Do not insert any aluminium parts. Follow the hygiene requirements for the reprocessing of medical devices (in Germany: RKI, KRINKO, MPBetreibV). Follow the instructions of the instrument and equipment manufacturers. For professional use only.

### Product range and specification

item number	container size	container type	packaging unit	PCN (each)	EAN/GTIN (each)	EAN/GTIN (packaging unit)
601490	2 L	canister	6 × 2 L	18313372	4260181252407	4260181252414
600091	5 L	canister	2 × 5 L	18313366	4260181253947	4260181251363
600071	10 L	canister	1 × 10 L	18313343	4260181253930	4260181251356

## Medical device class

IIb

## Composition

**The active ingredients contained in 100 g:**

1.125 g cocospropylene diammonium borate

## Storage and shelf life

**Shelf life:**

36 months

**Shelf life after opening**

12 months

Close the container carefully after each use.

Store the product in a well-ventilated place in a sealed condition and protect it from unauthorized access.

## Exposure times

Instru Star rfu spectrum of activity	condition (dirty/clean)	exposure time	norm
bactericidal	dirty	15 min.	EN 13727
yeastocidal			EN 14561 (DGHM)
tuberculocidal (M. terrae)			EN 13624
mycobactericidal (M. terrae, M. avium)			EN 14562 (DGHM)
active against enveloped viruses (Vaccinia strain Elstree)			EN 14348
virucidal effective against Polyoma SV40	10% FBS		EN 14563 (DGHM)
			EN 14348
			EN 14563 (DGHM)
			EN 14476
			DVV/RKI

## Information about this product



For further information, please visit our website **[www.drdeppe.de](http://www.drdeppe.de)**

For specific inquiries, please contact **[info@drdeppe.de](mailto:info@drdeppe.de)**