

Instru Suc

Product information

Product description

Instru Suc is a universal, economical, low-foaming and aldehyde-free concentrate for disinfecting suction units.

Area of application

Instru Suc is used for disinfecting suction units with and without amalgam separators.

Spectrum of activity

- bactericidal (including MRSA), yeasticidal

Instructions for use

Biofilm prevention

Daily disinfection with a 2% working solution can effectively prevent biofilm formation.

Biofilm removal

Existing biofilms can also be removed with Instru Suc: After 16 hours of contact time (overnight) with a 2% working solution, no germs were detected for an existing biofilm.

For professional use only.



Product range and specification

item number	container size	container type	packaging unit	PCN (each)	EAN/GTIN (each)	EAN/GTIN (packaging unit)
601489	2 L	canister	6 × 2 L	18313389	4260181252421	4260181252438
600047	5 L	canister	2 × 5 L	18313395	4260181250700	4260181251295

Medical device class

IIb

Composition

The active ingredients contained in 100 g:
7.5 g Dioctyldimethylammoniumchloride

Storage and shelf life

Shelf life:

36 months

Shelf life after opening:

12 months

Close the container carefully after each use.

Store the product in a well-ventilated place in a sealed condition and protect it from unauthorized access.

Exposure times

Instru Suc spectrum of activity	condition (dirty/clean)	exposure time	concentration	norm
bactericidal	dirty	15 min.	2%	EN 13727 (VAH) EN 14561 (VAH 15)
yeastocidal	dirty			EN 13624 (VAH) EN 14562 (VAH 15)

Information about this product



For further information,
please visit our website
www.drdeppe.de

For specific inquiries,
please contact
info@drdeppe.de