

Spray In QF

Product information

Product description

Spray In QF is a ready-to-use, liquid, alcohol based, aldehyde and QAV-free surface disinfectant.

Area of application

For rapid disinfection of alcohol-resistant surfaces in patient environments, clinics, practices, outpatient departments, and care areas, as well as in industry and public facilities. Excluded are surfaces of instruments, semicritical and critical medical devices. Spray In QF is also suitable for individuals in the

food sector, certified as HACCP-compliant according to EU Regulation (EC) No. 852/2004, and can be safely used within the framework of an HACCP concept.

Spectrum of activity

- bactericidal (incl. MRSA), yeasticidal
- tuberculocidal, mycobactericidal
- active against enveloped viruses (incl. HBV, HIV, HCV, vaccinia-, influenza- and corona-viruses)
- limited spectrum of virucidal activity (incl. noro-, rota-, adeno-viruses)

Instructions for use

The ready-to-use product is preferably to be used in the wiping method using a disposable wipe. Spray disinfection should preferably be limited to areas that are not reachable by wiping disinfection. When using the spray method, it should be combined with wiping method. It is important to keep the surface to be disinfected moist throughout the entire contact time to achieve effectiveness.



Product range and specification

item number	container size	container type	packaging unit	PCN (each)	EAN/GTIN (each)	EAN/GTIN (packaging unit)
601356*	500 ml	Flasche	18 × 500 ml	18313082	4260181254043	4260181251196
600917	1 L	Keulenflasche	12 × 1 L	18313047	4260181254012	4260181251165
601106	5 L	Kanister	2 × 5 L	18313053	4260181254029	4260181251172
601107	10 L	Kanister	1 × 10 L	18313076	4260181254036	4260181251189

* item is a biocidal product

Spray In QF

Product information

Medical device class

IIa (except for item no. 601356 - biocidal product only)

Biocide registration number

N-73614

Use disinfectant safely. Always read the label and product information before use.

Composition

The active ingredients contained in 100 g:

50 g Ethanol

Storage and shelf life

Shelf life:

36 months

Shelf life after opening:

12 months

Close the container carefully after each use.

Store the product in a well-ventilated place in a sealed condition and protect it from unauthorized access.

Exposure times

Spray In QF spectrum of activity	condition (dirty/clean)	exposure time	norm
bactericidal	with mechanical action clean	1 minute	EN 13727 EN 1276 EN 16615*
yeasticidal	with mechanical action clean	1 minute	EN 13624 EN 1650 EN 16615*
active against enveloped viruses (Modified Vaccinia Ankara)	with mechanical action clean	1 minute	EN 14476, virus-4-field-test**
bactericidal	without mechanical action clean	1 minute	EN 13727 EN 1276 EN 13697
yeasticidal	without mechanical action clean	1 minute	EN 13624 EN 1650 EN 13697
active against enveloped viruses (Modified Vaccinia Ankara)	without mechanical action clean	1 minute	EN 14476 EN 16777
limited spectrum of virucidal activity (Adenovirus type 5, Murine Norovirus)	clean	1 minute	EN 14476
tuberculocidal (M. terrae)	clean	1 minute	EN 14348
mycobactericidal (M. terrae und M. avium)	dirty	5 minutes	EN 14348

* tested on reference wipe

** according to EN 16615

Information
about this product



For further information,
please visit our website
www.drdeppe.de

For specific inquiries,
please contact
info@drdeppe.de

CE 0044



Laboratorium Dr. Deppe GmbH
Hooghe Weg 35 · 47906 Kempen
T +49 (0) 2152 5565-0 W www.drdeppe.de