

Spray Off wipes

Product information

Product description

Spray Off wipes are ready-to-use, alcohol-based, aldehyde-free surface disinfectant wipes. The wipe material is made from cellulose fibres, which are produced from the renewable raw material wood.

Area of application

For surface disinfection of alcohol-resistant surfaces in the patient-near environment, clinics, practices, outpatient clinics and care facilities, as well as in industry and public facilities. Surfaces of instruments resp. semi-critical and critical medical devices are excluded.

Spectrum of activity

- bactericidal (incl. MRSA), yeasticidal
- tuberculocidal, mycobactericidal
- fungicidal
- active against enveloped viruses (incl. HBV, HIV, HCV, vaccinia-, influenza- and corona-viruses)
- limited spectrum of virucidal activity (incl. noro-, rota-, adeno-viruses)
- virucidal effective against Polyoma SV-40
- virucidal

Instructions for use

The ready-to-use product is used by wiping. It is essential to keep the surface to be disinfected moist throughout the entire exposure times to achieve effectiveness.



Product range and specification

item number	container size	wipe size	container type	packaging unit	PCN (each)	EAN/GTIN (each)	EAN/GTIN (packaging unit)
601798	80 wipes	20 × 20 cm	flowpack (with LID)	12 × 80 wipes	19691751	4260181257464	4260181257471
601805	70 wipes	29 × 30 cm	bucket	1 × 70 wipes	19691745	4260181257488	4260181257495
601806	70 wipes	29 × 30 cm	bucket-refill pack	6 × 70 wipes	19692561	4260181257440	4260181257457

Spray Off wipes

Product information

Medical device class

IIa

Biocide registration number

N-91346

Use disinfectant safely. Always read the label and product information before use.

Composition

The active ingredients contained in 100 g soaking solution:

63.50 g Ethanol

0.05 g Didecyldimethylammonium chloride

Storage and shelf life

Shelf life:

36 months

Shelf life after opening:

28 days

Close the container carefully after each use. Only use moist wipes.

Store the product in a well-ventilated place in a sealed condition and protect it from unauthorized access.

Exposure times

Spray Off wipes (Cellulose) spectrum of activity	condition (dirty/cleab)	exposure time	norm
bactericidal	with mechanical action dirty	1 minute	EN 13727* EN 16615
yeasticidal	with mechanical action dirty	1 minute	EN 13624* EN 16615
tuberculocidal (M. terrae), mycobactericidal (M. terrae, M. avium)	dirty	1 minute	EN 14348*
fungicidal (C. albicans, A. brasiliensis)	dirty	5 minutes	EN 13624*
active against enveloped viruses Vaccinia strain Elstree, BVDV strain NADL)	10% FBS	1 minute	DVV/RKI*
active against enveloped viruses (Modified Vaccinia Ankara)	clean	1 minute	EN 14476*
limited spectrum of virucidal activity (Murine Norovirus, Adenovirus type 5)	10% FBS	1 minute	DVV/RKI*
virucidal effective against Polyoma SV-40	10% FBS	5 minutes	DVV/RKI*
virucidal (Murine Norovirus, Adenovirus type 5, Poliovirus type 1)	clean	5 minutes	EN 14476*

* tested on soaking solution

Information
about this product



For further information, please visit our website www.drdeppe.de

For specific inquiries, please contact info@drdeppe.de

CE 0044



Laboratorium Dr. Deppe GmbH
Hooghe Weg 35 · 47906 Kempen
T +49 (0) 2152 5565-0 W www.drdeppe.de