

# Spray On wipes

Product information

## Product description

Spray On wipes are ready-to-use alcohol-based surface disinfectant wipes that are free from aldehydes and QAVs. The wipe material is made from cellulose fibres, which are produced from wood, a renewable raw material.

## Area of application

Health care facilities and pharmaceutical and cosmetic industry, for example patient-near environments, working areas/desks, general equipment (excluding food contact surfaces): disinfection of small hard/non-porous surfaces.

## Spectrum of activity

- bactericidal (incl. MRSA), yeasticidal
- active against enveloped viruses (incl. HBV, HIV, HCV, vaccinia-, influenza- and corona-viruses)

## Instructions for use

The surface to be disinfected is wiped and moistened with a sufficient amount of the product to ensure complete wetting. It is important to note that the surface to be disinfected must be kept moist throughout the entire exposure time in order to achieve effectiveness.



The biocidal product Spray On wipes (active ingredients: Propan-1-ol and Propan-2-ol) has received Union and Swiss authorizations for product type 2 disinfectants and algacides not intended for direct application to humans or animals) and product-type 4 (food and feed area) with authorization numbers EU-0027668-0000 and CH-0031680-0000. The Union authorization is valid until July 31, 2032.

## Product range and specification

item number	container size	wipe size	container type	packaging unit	PCN (each)	EAN/GTIN (each)	EAN/GTIN (packaging unit)
601212	80 wipes	20 × 20 cm	flowpack (with LID)	12 × 80 wipes	18312361	4260181253879	4260181251318
601214	70 wipes	29 × 30 cm	bucket	1 × 70 wipes	18312384	4260181253893	4260181251332
601213	70 wipes	29 × 30 cm	bucket-refill pack	6 × 70 wipes	18312378	4260181253886	4260181251325

## Union authorisation number

EU-0027668-0000

## Swiss authorisation number

CH-0031680-0000

Use disinfectant safely. Always read the label and product information before use.

## Composition

### The active ingredients contained in 100 g soaking solution:

45 g Propan-1-ol

15 g Propan-2-ol

## Storage and shelf life

### Shelf life:

flowpack 24 months,

bucket and bucket refill pack 36 months

### Shelf life after opening:

28 days

Close the container carefully after each use.  
Only use moist wipes.

Store the product in a well-ventilated place in a sealed condition and protect it from unauthorized access.

## Exposure times

Spray On wipes (Cellulose) spectrum of activity	condition (dirty/clean)	exposure time	norm
bactericidal	with mechanical action dirty	1 minute	EN 13727* EN 16615
yeasticidal	with mechanical action dirty	1 minute	EN 13624* EN 16615
Active against enveloped viruses (Modified Vaccinia Ankara)	dirty	1 minute	EN 14476*

\* tested on soaking solution

## Informationen about this product



For further information,  
please visit our website  
[www.drdeppe.de](http://www.drdeppe.de)

For specific inquiries,  
please contact  
[info@drdeppe.de](mailto:info@drdeppe.de)